**L4-08-4613 rev. B 12/11/2023 Neurostimulation Electrodes –
Instructions for Use - Translations**

English (EN)

Instructions for Application, Operation and Removal:

1. Skin must always be clean, dry and free from lotion.
2. Do not apply to broken skin.
3. Attach lead wires to electrodes.
4. Remove the protective liner from the electrode. Save liner.
5. Apply electrode securely to skin. Apply center first, then smooth down to electrode edges.
6. Attach lead wires to stimulator device. Turn stimulator on; adjust according to stimulator instructions for treatment. Do not exceed an intensity of 0.1 Watt/cm².
7. When stimulation session is complete, turn stimulator **OFF**.
8. Lift at the edge and peel to remove electrodes from skin.

Instructions for Electrode Care and Storage:

9. Replace the saved protective liner on the electrode. Be sure to place the **ON** side against the electrode gel surface.
10. Disconnect the stimulator wires from the electrode lead wire connectors.
11. Replace electrodes in the original package.
12. If electrodes begin losing adhesion, gently rubbing one or two drops of water onto gel surface **may** extend usage. If not, replace with new electrodes.

Caution: Electrodes not firmly attached to the skin greatly increase the chance of burns and other adverse reactions.

Warnings and Precautions:

- ⚠ For all relevant hazards, contraindications, warnings, precautions and adverse side effects, refer to the stimulator device user's manual.
- Keep electrodes out of the reach of children.
 - Electrodes are intended for single person use only.
 - DO NOT remove electrode by pulling on lead wire.
 - DO NOT use electrodes that are the wrong size or shape for your device, or incorrectly applied. This could result in skin irritation or thermal burns.
 - Keep electrodes separated during treatment.
 - DO NOT apply to broken skin. Should a skin rash or skin burn occur, immediately discontinue use and contact your physician.
 - DO NOT exceed an intensity of 0.1 Watt/cm².
 - DO NOT apply electrodes across the head, or across the heart, or on the front of the neck.
 - DO NOT stimulate while driving or operating machinery.

NOTE: The life of the electrode varies depending on skin conditions, skin preparation, type of stimulation, storage and climate. Please read the stimulator device user's manual thoroughly before use.

Serious events experienced from using this product should be reported immediately to Axelgaard Manufacturing Co. Ltd. and the local competent authority

L4-08-4613 rev. B 12/11/2023 Neurostimulation Electrodes –
Instructions for Use - Translations

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